**3M ESPE
Dental Products**

St. Paul, MN 55144-1000

3M, ESPE, Clicker, Vitremer, Scotchbond, Adper, Z100 and Sof-Lex are trademarks of 3M or 3M ESPE AG.

© 3M 2006. All rights reserved.

**3M ESPE AG
Dental Products**

D-82229 Seefeld – Germany

44-0007-4427-4-A

3M ESPE

F2000 Compomer Restorative System

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F2000 compomer restorative contains methacrylates. F2000 compomer primer/adhesive also contains methacrylates including HEMA (2-hydroxyethylmethacrylate). A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resin should be avoided. **Use of protective gloves and a no-touch technique is recommended.** If compomer restorative or compomer primer/adhesive

contacts skin, wash immediately with soap and water. Acrylates may penetrate commonly used gloves. If compomer restorative or compomer primer/adhesive contacts glove, remove and discard glove, wash hands immediately with soap and water and then reglove. Uncured compomer restorative may cause mild irritation to the oral soft tissue. **Avoid contact with oral soft tissue.** If accidental contact occurs, remove uncured paste and rinse tissue with water. Compomer restorative or compomer primer/adhesive may cause eye irritation upon contact. Avoid contact with eyes. If accidental contact occurs, flush immediately with large amounts of water. If irritation persists, consult a physician.

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For core buildups, the blue shade will provide

contrasting color to tooth structure. All compomer restorative shades may be used for core buildups.

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6. Compomer primer/adhesive application. Clicker Dispensing System operation (Also

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


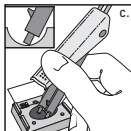
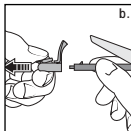
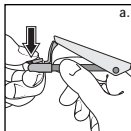
d.) Fully depress dispenser lever to dispense material from each side of cartridge into well. Release lever.

e.) Remove dispenser. Wipe the cartridge tips with an alcohol-dampened gauze.

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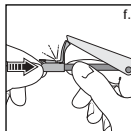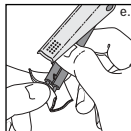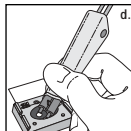Using an applicator brush, mix dispensed compomer primer/adhesive and **generously** apply it to all enamel and dentinal surfaces to be bonded. For core buildups

where pins have been placed, apply compomer primer/adhesive to pins as well.

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Syringe. Dispense the required amount of compomer restorative from the syringe onto a

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9. Finishing. The compomer restorative can be finished using conventional rotary instruments immediately after light curing. The Sof-Lex™ Finishing and Polishing system, manufactured by

3M ESPE, and Sof-Lex Strips, manufactured for 3M ESPE, are recommended for polishing.

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2. Compomer primer/adhesive application. Apply F2000 compomer primer/adhesive to enamel and dentin to be bonded with the F2000 compomer restorative. Wait 30 seconds, then gently air dry

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5. Etching. Apply Scotchbond™ etchant, manufactured by 3M ESPE, to the remaining enamel and dentin and to the occlusal portion of the compomer restorative. Wait 15 seconds. Rinse and dry. Refer to separate etchant instructions for full instructions and precautions for the product.

6. Adhesive application. Apply a 3M ESPE

adhesive/adhesive system to the etched surfaces, dry and light cure according to individual product instructions. Adper™ Scotchbond™ 1 Adhesive System, manufactured by 3M ESPE, or the Adper™ Scotchbond™ Multi-Purpose Adhesive System, manufactured by 3M ESPE, can be used. Refer to the individual adhesive system instructions for full instructions and precautions for the products.

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Storage and Use

1. Do not store the products in proximity to eugenol-containing materials.
2. Do not expose the products to elevated temperature or intense light.
3. F2000 compomer restorative capsules are

sensitive to humidity and should always be stored in their foil pouches. Do not remove capsule from foil pouch until just prior to use.

4. The products are designed to be used at room temperatures of 21 - 24°C or 70 - 75°F.

5. Shelf life of the compomer restorative at room temperatures is 36 months. Shelf life of the compomer primer/adhesive in the Clicker Dispensing System is 24 months. See outer package for expiration dates.

6. An approximate number of compomer primer/adhesive applications remaining in the Clicker Dispensing System is indicated on the underside of the dispenser by the number on the plunger adjacent to the black arrow on the cartridge.

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Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M ESPE

F2000 Compomer Restorative System

F2000 Compomer Restorative with F2000 Compomer Primer/Adhesive in Clicker™ Dispensing System

F2000 Compomer Restorative with F2000 Compomer Primer/Adhesive in Clicker™ Dispensing System

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**3M ESPE
Dental Products**
44-0007-4428-2-A

General Information

The F2000 compomer restorative system, manufactured by 3M ESPE, is comprised of the F2000 compomer restorative filling material and the F2000 compomer primer/adhesive used for bonding the compomer restorative to tooth structure.

F2000 compomer restorative is a one-part, light-curable, fluoride-releasing, radiopaque paste that has excellent handling characteristics and high physical properties. It is packaged in both single-use capsules and multi-dose syringes.

F2000 compomer primer/adhesive is a two-part product; one part is similar in composition to the commercially available Vitremer™ primer, manufactured by 3M ESPE, the second part is a five percent aqueous maleic acid solution. The materials are contained in the unique Clicker™ Dispensing

System, manufactured by 3M ESPE, which provides simultaneous dispensing of each component for easy, convenient delivery. The compomer primer/adhesive does not require a separate acid etch step for bonding.

Indications

F2000 compomer restorative system is indicated for:

- Class III cavities
- Class V cavities and cervical erosion/abrasion lesions
- Root caries lesions
- Class II laminate or open sandwich technique
- Class I and Class II cavities in primary teeth
- Temporary repair of fractured teeth
- Core buildup
- Class I and II restorations in permanent teeth, provided the width of the cavities is less than two thirds of the intercuspal distance

Precautions for Dental Personnel and Patients

F2000 compomer primer/adhesive contains five weight percent maleic acid. Protective eye wear for patients and dental staff is recommended when using the compomer primer/adhesive. Avoid contact with oral soft tissue, eyes and skin. If accidental contact occurs, flush immediately with large amounts of water. For eye contact consult a physician.

F2000 compomer restorative contains methacrylates. F2000 compomer primer/adhesive also contains methacrylates including HEMA (2-hydroxyethylmethacrylate). A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resin should be avoided. **Use of protective gloves and a no-touch technique is recommended.** If compomer restorative or compomer primer/adhesive contacts skin, wash immediately with soap and

water. Acrylates may penetrate commonly used gloves. If compomer restorative or compomer primer/adhesive contacts glove, remove and discard glove, wash hands immediately with soap and water and then reglove. Uncured compomer restorative may cause mild irritation to the oral soft tissue. **Avoid contact with oral soft tissue.** If accidental contact occurs, remove uncured paste and rinse tissue with water. Compomer restorative or compomer primer/adhesive may cause eye irritation upon contact. Avoid contact with eyes. If accidental contact occurs, flush immediately with large amounts of water. If irritation persists, consult a physician.

General Instructions for Use

1. Shade selection. Select the desired compomer restorative shade using the F2000 compomer restorative shade guide.

For core buildups, the blue shade will provide contrasting color to tooth structure. All compomer

restorative shades may be used for core buildups.

2. Isolation. Rubber dam is the preferred method of isolation. Gingival retraction and cotton rolls may be used.

3. Cavity preparation. Remove caries. Prepare cavity with minimal tooth reduction and with rounded internal line angles. If no cavity preparation is required, it is important to clean the surfaces to be restored using a plain pumice/water slurry. For core buildups, multiple missing cusps may require placement of pins for retention prior to bonding. Rinse and dry cavity.

4. Pulp protection. If there is no exposure of the pulp, no liner is required.

5. Matrix placement. If needed, place a matrix appropriate for the restoration.

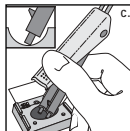
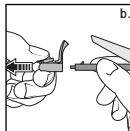
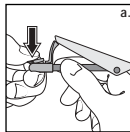
6. Compomer primer/adhesive application. Clicker Dispensing System operation (Also see graphic instructions.)

a.) Press and hold dispenser cap lever to unlock cap.

b.) Slide cap off of dispenser.

Do not depress dispenser lever during cap removal.




c.) Touch cartridge tips to bottom of well. **Physical contact of the cartridge tips to the well is critical to assure proper dispensing of materials.**

d.) Fully depress dispenser lever to dispense material from each side of cartridge into well. Release lever. e.) Remove dispenser. Wipe the cartridge tips with an alcohol-dampened gauze.

f.) Replace cap by holding the sides of the cartridge and sliding cap into place until securely latched as indicated by an audible "click." **Do not depress dispenser lever during recapping.**

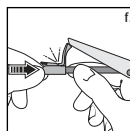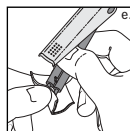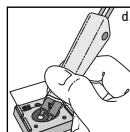Using an applicator brush, mix dispensed compomer primer/adhesive and **generously** apply it to all enamel and dentinal surfaces

to be bonded. For core buildups where pins have been placed, apply compomer primer/adhesive to pins as well.

Wait 30 seconds. Replenish compomer primer/adhesive as needed to keep the surfaces wet for the recommended wait time.

Gently dry the compomer primer/adhesive for 5 - 10 seconds to leave a thin film. The treated surfaces should appear coated and shiny.

Light cure the compomer primer/adhesive for 10 seconds using a high-intensity light such as a 3M curing light.

7. Compomer restorative placement. F2000 compomer restorative is available in both single-use capsules and multi-dose syringes.

SINGLE DOSE CAPSULE. (See separate 3M ESPE Delivery System, manufactured for 3M ESPE, Instructions).

Syringe. Dispense the required amount of compomer restorative from the syringe onto a dispensing pad.

Place the compomer restorative in increments and light cure as indicated in the following section 8.

8. Curing. F2000 compomer restorative will cure only by exposure to light. Cure each increment by exposing its entire surface area to high-intensity light such as that from a 3M ESPE curing light. Hold the light guide tip as close to the compomer restorative as possible during light exposure. The recommended exposure time and maximum increment thickness for each shade is shown below.

Shade	Thickness	Exposure Time
Pedo, A2, A3, B2, B3, C2, Blue	2.5 mm	40 secs.
A3.5, A4	2.0 mm	40 secs.

9. Finishing. The compomer restorative can be finished using conventional rotary instruments immediately after light curing. The Sof-Lex™ Finishing and Polishing system, manufactured by 3M ESPE, and Sof-Lex Strips, manufactured for 3M ESPE, are recommended for polishing.

Note for core buildups:

- The compomer restorative is compatible with commonly-used impressioning materials.
- The compomer restorative should be kept wet with saliva or lubricated to prevent bonding to chemical-cure provisionals.
- Commonly-used temporary luting cements will not bond to the compomer restorative.

Instructions for Class II Laminate or Open Sandwich Technique

1. Review. Review the above General Instructions for Use prior to placing a laminate restoration.

2. Compomer primer/adhesive application. Apply F2000 compomer primer/adhesive to enamel and dentin to be bonded with the F2000 compomer restorative. Wait 30 seconds, then gently air dry 5 - 10 seconds. Light cure for 10 seconds.

3. Compomer restorative placement. F2000 compomer restorative will cure only by exposure to light. Place the compomer restorative incrementally extending it no further than just apical to the proximal contact point and light cure.

4. Margin and cavity refinement. Loosen and protect the matrix. Using a rotary instrument, remove any F2000 compomer primer/adhesive and compomer restorative from the enamel margins and cavity walls remaining to be bonded. Rinse and dry cavity. Reposition matrix.

5. Etching. Apply Scotchbond™ etchant, manufactured by 3M ESPE, to the remaining enamel and dentin and to the occlusal portion of the compomer restorative. Wait 15 seconds. Rinse and

dry. Refer to separate etchant instructions for full instructions and precautions for the product.

6. Adhesive application. Apply a 3M ESPE adhesive/adhesive system to the etched surfaces, dry and light cure according to individual product instructions. Adper™ Scotchbond™ 1 Adhesive System, manufactured by 3M ESPE, or the Adper™ Scotchbond™ Multi-Purpose Adhesive System, manufactured by 3M ESPE, can be used. Refer to the individual adhesive system instructions for full instructions and precautions for the products.

7. Composite placement. Use Z100™ MP Restorative, manufactured by 3M ESPE, to complete the restoration. Refer to separate Z100 MP Restorative instructions for full instructions and precautions for the product.

Storage and Use

1. Do not store the products in proximity to eugenol-containing materials.

2. Do not expose the products to elevated temperature or intense light.
3. F2000 compomer restorative capsules are sensitive to humidity and should always be stored in their foil pouches. Do not remove capsule from foil pouch until just prior to use.
4. The products are designed to be used at room temperatures of 21 - 24°C or 70 - 75°F.
5. Shelf life of the compomer restorative at room temperatures is 36 months. Shelf life of the compomer primer/adhesive in the Clicker Dispensing System is 24 months. See outer package for expiration dates.
6. An approximate number of compomer primer/adhesive applications remaining in the Clicker Dispensing System is indicated on the underside of the dispenser by the number on the plunger adjacent to the black arrow on the cartridge.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental

or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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