ENGLISH

Filtek™ LS Low Shrink Posterior Restorative, manufactured by 3M ESPE, is a light-curing, radiopaque, silorane-based composite for the posterior dental area.

Product Description

Filtek™ LS Low Shrink Posterior Restorative, manufactured by 3M ESPE, is a light-curing, radiopaque, silorane-based composite for the posterior dental area. It contains a hydrophobic resin matrix and is characterized by very low polymerization shrinkage and low water absorption. Filtek LS Low Shrink Posterior Restorative filling material is available in the shades A2, A3, B2, and C2, and can be purchased in syringes for repeated dosing and capsules for one-time dosing. It contains 55% volume (76% weight) inorganic fillers with a particle size between 0.1 and 2 µm.

LS System Adhesive, manufactured by 3M ESPE, consists of a self-etching primer and a bond in dosing bottles. Thanks to its special structural viscosity, LS Bond flows ideally onto the cavity walls during application and can be distributed very evenly in the cavity by the use of the applicator. The Bond stops flowing as soon as it loses contact with the applicator or when the blowing is stopped, preventing pooling, uncontrolled flow, and uneven layer thicknesses.

The filling material and adhesive system are polymerized by the use of halogen or LED light-curing devices; see Exposure Times/Devices.

Filtek LS and LS System Adhesive may be used only in combination with one another as a system. LS System Adhesive may not be used as primer and/or bond for other filling materials. Filtek LS may not be combined with other bonding systems.

- Please see the pertinent Instructions for Use for details on all of the products mentioned below. These Instructions for Use must be retained for reference during the entire time that the product is being used.

Indications

- Direct posterior dental restoration of the cavity classes I and II

Precautions

For Patients

Filtek LS and LS System Adhesive contain substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of Filtek LS in patients with known silorane (silicone-based epoxies) allergies and LS System Adhesive in patients with known acrylate allergies.

If prolonged contact with oral tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use.

For Dental Personnel

Filtek LS and LS System Adhesive contain substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water.

The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If the product contacts the

glove, remove and discard the glove, wash the hands immediately with soap and water and then re-glove.

If allergic reaction occurs, seek medical attention as needed.

LS System Adhesive Primer is a flammable material.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparatory Measures

- Use the Filtek LS shade guide to select the matching composite color.

Preparation

- Prepare the cavity according to standard adhesive technique procedures.
- Use a water spray to remove any loose preparation particles.
- When LS System Adhesive is applied to unprepared tooth structure, clean the tooth structure by the usual methods (e.g., with oil-free prophy paste or a air-abrasion procedure) and rinse thoroughly with a water spray. Etch for 15 seconds with phosphoric acid, then rinse thoroughly again.
- Blow the cavity dry with 2-3 brief bursts of dry, oil-free air, or use cotton wool pellets to pat it dry. Do not overdry!

- Avoid contamination of any kind from blood, saliva, or sulcus fluid. We recommend using a rubber dam to ensure that the area remains adequately dry.

Pulp Protection

To prevent irritation of the pulp, any areas in the proximity of the pulp should be covered punctiformly with a firm-setting calcium hydroxide preparation (e.g., Alkaliner™, manufactured by 3M ESPE).

Cavity Liner / Reliner

Self-adhesive materials such as glass ionomer cements (e.g., Ketac™ Bond, manufactured by 3M ESPE) or resin-modified glass ionomer cements (e.g., Vitrebond™ manufactured by 3M ESPE) may be used as cavity liner or reliner. Composites and compomers (including flowables) which are bonded to the tooth structure using an adhesive may not be used.

Position Matrix

- When filling Class II, a pre-contoured metal or plastic matrix or a pre-contoured metal matrix band must be positioned before application of the adhesive.

- Make sure there is good contact between the matrix and the proximal tooth.

- Wedge the matrix firmly in place in the sulcus area to avoid interdental composite overhanging.

Application LS System Adhesive - Self-Etch Primer

- Shake the bottle briefly before dosing so that the primer becomes less viscous and flows to the dosing tip.

- Place one drop of Primer into Depression 1 of the dosing well, then close the dosing well to protect the primer from light and prevent the evaporation of the solvent.

- Close the bottle immediately after use; do not interchange the caps for the Primer and Bond.

- Use the enclosed black disposable applicator to apply the Primer to the entire surface of the cavity and massage over the entire area for 15 sec.

- Protect the solution from intensive light during treatment.

- If the applicator becomes contaminated with blood, saliva, or sulcus liquid, use a new 3M ESPE disposable applicator. 3M ESPE disposable applicators can be ordered separately. A size M (Medium) is recommended for the application of the LS System Adhesive.

- Use a gentle stream of air until the Primer is spread to an even film and does not move any longer. Fill in any gaps on the edge.

- Cure the Primer for 10 seconds using a halogen or LED light-curing device.

Application LS System Adhesive - Bond

- Shake the bottle briefly before dosing so that the Bond becomes less viscous and flows to the dosing tip.

- Place one drop of Bond in Depression 2 of the dosing well and close the dosing well to protect the Bond from light. Do not mix Primer and Bond.

- Close the bottle immediately after use; do not interchange the caps for the Primer and Bond.

- Use the enclosed green disposable applicator to apply the Bond to the entire area of the cavity.

- Protect the solution from intensive light during treatment.

- Use a gentle stream of air until the Bond is spread to an even film and does not move any longer. Fill in any gaps on the edge.

- Cure the Bond for 10 seconds using a halogen or LED light device.

- Immediately fill the cavity with Filtek LS.

Dosing of the Filling Material from the Syringe

- Remove the cap of the syringe.

- Slowly turn the syringe plunger handle clockwise and apply a sufficient quantity of the filling material to a mixing pad.

- Turn the plunger handle a half-turn counter-clockwise so that no further paste emerges and replace the syringe cap immediately.

Dosing of the Filling Material from the Capsule

Each capsule may be used for only one patient!

- Insert the capsule into a commercially available dispenser. Check the capsule for secure seating in the dispenser before application.

- Remove the cap and apply Filtek LS filling material directly into the cavity.

Application of the Filling Material

- Use a suitable metal or plastic instrument to shape the filling material exactly to the cavity. The thickness of the individual increments must not exceed 2.5 mm.

- Cure each increment individually with a halogen or LED light device, using only the standard mode of the light device at a constant light intensity. The exposure time depends on the device model and is shown in the section "Exposure Times/Devices".

- Hold the light guide tip as close as possible to the material being cured.

- When using a metal matrix, re-expose the corresponding filling area after removing the matrix.

Exposure Times/Devices

- Use full and constant light intensity to polymerize Filtek LS during the full exposure time.

- Filtek LS requires an exposure time of at least 20 seconds, which cannot be shortened by higher light intensities. To prevent overheating of the tooth during the minimum exposure time of 20 seconds, do not use very high light intensities such as those from plasma or laser devices.

Types of devices which may be used:

| | |
|---------------------------------|-----------------------|
| Halogen light devices | Exposure time |
| Wave length spectrum 400-500 nm | |
| Output 500 – 1400 mW/cm², | 40 sec, standard mode |

| | |
|---------------------------------|-----------------------|
| LED light devices | |
| Wave length spectrum 430-480 nm | |
| Output 500-1000 mW/cm², | 40 sec, standard mode |
| Output 1000 - 1500 mW/cm², | 20 sec, standard mode |

(e.g., Elipar™ FreeLight 2, manufactured by 3M ESPE)

Trimming

- After curing, trim and polish the restoration with appropriate diamond abrasives, polishing discs with progressively decreasing grit size, ultra-fine grain diamonds, and silicone polishers.

- Be sure to provide effective water cooling and adequate suction removal of the polishing dust!

Repair

- Conventional methacrylate composite and compomer fillings can be repaired using Filtek LS and LS System Adhesive as described in these instructions for use.

- Filtek LS fillings can be repaired with methacrylate composite and compomer filling materials and the appropriate adhesive. Please follow the instructions for use of the specific material.

Filtek LS as Base-filling

- Once Filtek LS base-fillings have been placed, methacrylate composite or compomer filling material can be layered on top. Use the adhesive appropriate for the methacrylate composite or compomer material and follow the instructions for use for the specific material.

Notes

- Use LS System Adhesive from the dosing well for one patient only.

- Filtek LS and LS System Adhesive also polymerize when exposed to daylight or artificial light. For this reason, do not dispense the products until immediately before beginning using, and avoid intensive lighting while working with the material. Close the bottles immediately after dispensing the material.

Disposal

Do not dispose of LS System Adhesive bottles until they have been completely emptied. Residues of the etching liquids – heavily thinned with water or neutralized – may be disposed of in waste water.

Storage and Stability

Protect LS System Adhesive bottles from direct light and store at temperatures between 2° and 8°C/36° and 46°F.

Store Filtek LS at 2-25°C/36-77°F. Filtek LS which has been stored below room temperature becomes more viscous and will be more difficult to extrude, especially from the capsule. To avoid this problem, allow the paste to stand at room temperature for a few minutes before pressing it out.

Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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3M, ESPE, Alkaliner, Ketac, Vitrebond, Elipar and Filtek are trademarks of 3M or 3M ESPE AG. © 3M 2007. All rights reserved.