Instructions for Use
Mode d’emploi
Información de uso

3M ESPE AG
Dental Products
D-82229 Seefeld – Germany

3M ESPE
Dental Products
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.

3M, ESPE, Garant, Impregum, Impresept, Penta, PentaMatic and Pentamix are trademarks of 3M or 3M ESPE AG. © 3M 2007. All rights reserved.

ENGLISH

Product Description

Impregum™ Penta™ Soft Quick Step Heavy Body, manufactured by 3M ESPE, is a fast-setting polyether impression material with a heavy-bodied consistency. The mixing ratio, based on volume, is 5 parts base paste : 1 part catalyst. The paste is mixed in Pentamix versions from Pentamix™ 2 (September 1999) on, earlier Pentamix units (before September 1999) extrude the material too slow. Pentamix mixing units are manufactured by 3M ESPE.

Each of the foil bags is sealed with a **PentaMatic™** cap, manufactured by 3M ESPE. This **PentaMatic** cap opens the foil bag automatically in the Pentamix mixing unit as soon as enough pressure is built up by the plunger.

Impregum™ Soft Quick Step Light Body is a fast-setting polyether impression material with a light-bodied consistency in the Garant™ cartridge, both products manufactured by 3M ESPE. The mixing ratio, based on volume, is 2 parts base paste : 1 part catalyst.

Due to the low Shore hardness, the impression materials are highly suitable for impressions with slight undercuts, even without additional blocking out.

See the pertinent information for use for details on the Pentamix mixing unit and accessories, Polyether Adhesive, Penta™ Mixing Tips Red, Garant™ Dispenser, Garant™ Mixing Tips and Intra-oral Tips, and Impresept™, all products manufactured by or for 3M ESPE.

☞ These instructions for use must be kept for the duration of product use.

Range of Indications

- Impressions of inlay, onlay, veneer, crown, and bridge preparations
- Fixation and implant impressions

Impregum Penta Soft Quick Step Heavy Body and Impregum Soft Quick Step Light Body are especially suitable for taking impressions of single-unit and double-unit preparations.

Precautionary Measures

If an adverse reaction is observed in a patient, discontinue use of the product in this patient.

3M ESPE MSDSs can be obtained from www.mmm.com or contact your local subsidiary.

Preparatory Steps

- Block-out undercuts or areas where gingival recession exists to prevent the impression material from "locking" onto the tooth structure. Failure to block-out may make tray removal difficult or cause extraction of natural teeth or prosthesis. The block-out is especially important when using impression materials with high Shore hardness.

Impression tray: Any impression trays generally used for precision impressions are suitable.

- For sufficient adhesion to metal and plastic, apply a thin layer of Polyether Adhesive to the impression tray and allow it to dry completely (at least 30–60 sec, 15 min is ideal).

Pentamix/Penta Cartridge/Foil Bag:

- Use the Impregum Penta Soft Quick Step Heavy Body foil bag only in combination with the designated Penta cartridge and with Penta Mixing Tips Red.
- Prepare new foil bags for use before taking the first impression by extruding about 3 cm of paste. The paste must emerge in a uniform color.
- If a new mixing tip has already been mounted when the cartridge is inserted, make sure that the drive shaft engages the mixing tip prior to start of mixing.

Garant Dispenser/Cartridge:

- At the start of a new cartridge and before every new mixing process, first extrude a quantity of paste without using a mixing tip until both components emerge evenly; remove any paste plugs that have formed in the cartridge openings.
- Then attach a Garant Mixing Tip White and a Garant Intra-oral Tip White.

Retraction

Solutions based on aluminum hydroxide chloride or aluminum sulfate are suitable retraction agents. The use of hemostatic threads or rings containing epinephrine (adrenaline), 8-hydroxyquinoline sulfate, and iron (III) sulfate for retraction may impair the setting of polyether impression materials.

- Keep the area dry where the impression is being taken.
- If necessary, use hemostatic threads or rings for subgingival preparations.
- Before taking the impression, remove completely any residue of the retraction agent by rinsing and drying.

Dosing and Mixing

- Dosing and mixing is done in the Pentamix mixing unit or in the Garant Dispenser, respectively. Only the Pentamix 2 or younger Pentamix versions must be used to mix Impregum Penta Soft Quick Step Heavy Body.

Impression material	Processing time from start of mixing* min:sec	Residence time in the mouth min:sec	Setting time from start of mixing* min:sec
Impregum Penta Soft Quick Step Heavy Body	Max. 1:00 ¹	+ 3:00	= Max. 4:00
Impregum Soft Quick Step Light Body	Max. 1:00 ²	+ 3:00	= Max. 4:00

* Start of mixing = Entry of paste into the Penta Mixing Tip Red or Garant Mixing Tip

¹ Time available to load the impression tray and position it in the mouth

² Time available for application around the preparation and, as necessary, prior filling of the Elastomer Syringe.

The residence time in the mouth must always be 3 minutes, regardless the previously needed processing time.

Impression Taking

One-step Technique

- Load the impression tray which has been prepared with adhesive with Impregum Penta Soft Quick Step Heavy Body. Keep the Penta Mixing Tip Red immersed in the impression material at all times.
- Apply Impregum Soft Quick Step Light Body around the dried preparation from the bottom up. Keep the tip of the intra-oral tip immersed in the impression material and in contact with the tooth surface at all times during application.
 - Application around the preparation and loading of the impression tray must be coordinated so that the loaded impression tray can be placed in the mouth immediately after the application. The processing times shown for the materials used must not be exceeded.** Otherwise, the two materials will not set up together. The consequence would be deformations in the impression or a poor bond of the two materials.
- Position the loaded impression tray in the mouth immediately after the application and hold it in place, without applying pressure, until setting is complete.
- When the setting time has lapsed, remove the impression from the mouth.
 - To remedy initial adhesion ("setting the valve"), especially with upper jaw impressions, remove the tray from the gingiva on one side in a posterior position. If this proves difficult it may be necessary to carefully blow some air or water between the impression and the gingiva.

After Impression Taking

- Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene

- Place the impression in a standard disinfectant solution commonly used for impressions, e.g. Impresept™*, for the period of time recommended by the manufacturer, i. e., 10 min in the case of Impresept. Excessive disinfection may damage the impression.

- After disinfection, rinse the impression under running water for approx. 15 sec, then dry.

* Impresept is not available in all countries.

Model Preparation

- Prepare a cast from the impression with a commercial specialized stone plaster no earlier than 30 min and no later than 14 days after taking the impression.

- To avoid introducing bubbles into the model, briefly pre-rinse with water and dry with air. Do not use surfactants as these impair the quality of polyether impressions and are not necessary.
- Polyether impressions may be silver-coated, whereas copper-coating is not possible.

Cleaning

- Garant Dispenser:** Remove paste that has not set using an alcohol-soaked cloth. The dispenser handle and plunger can be autoclaved up to a temperature of 135°C/275°F. Disassemble the dispenser before autoclaving. Glutaraldehyde-based solutions can be used for disinfection.
- Impression Tray:** The adhesive can be removed from impression trays using acetone.

Notes

- 3M ESPE polyether materials may be combined only with 3M ESPE polyethers. The working and setting times of the individual products must be strictly observed.

Impregum Penta Soft Quick Step Heavy Body:

- At temperatures below 18°C/64°F, the viscosity of the pastes will increase to an extent that there may be mixing problems in the unit. Store the pastes for 1 day at a minimum temperature of 18°C/64°F; they can then be processed without loss of quality.

Impregum Soft Quick Step Light Body:

- Turning or reattaching a mixing tip without subsequent impression taking may lead to carry-over of paste and clogging. The filled mixing tip should therefore be left as a closure on the Garant cartridge until the next impression is taken.
- If the material is stored at temperatures below 18°C/64°F, precise dosing and mixing can no longer be guaranteed even if subsequently kept at room temperature for a longer period of time.

- Direct exposure to sunlight and damp storage conditions may damage the impression, see Storage.
- Polyether impressions should not be exposed to solvent-containing liquids under any circumstances. Swelling and imprecise modeling may result.

Storage and Stability

Store the products at 18–25°C/64–77°F. **Do not refrigerate!** Do not use after the expiration date. Store impressions in a dry place away from light at temperatures below 30°C/86°F.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manu- facture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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