3M ESPE
Ketac™ Silver
Silver-Reinforced Glass Ionomer Restorative Material
Ⓛ Silberverstärktes Glasionomer Füllungsmaterial
ⓕ Matériau d’obturation en verre ionomère à renfort d’argent
Ⓛ Materiale per otturazioni vetro-ionomero rinforzato all’argento
Ⓔ Material de obturación de vidrio ionómero reforzado con plata
Ⓟ Material de obturacão de vidro ionómero reforçado com prata
ⓃL Met zilver versterkt glasionomeer vulmateriaal
ⒸR Υαλοϊονομερήs κοπία για εμφράξεις ενισχυμένη με άργυρο
Ⓢ Silberförstärkt glasjonomer fyllningsmaterial
ⓂIM Hopeavahvisteinen lasi-ionomeeritäytealne
ⓃK Sølvforstærket glasionomer-fyldningsmateriale
Ⓝ Sølvforsterket glassionomer fyllingsmateriale

Instructions for Use	Gebbruksinformatie
Gebrauchsinformation	Οδηγίες χρήσης
Mode d’emploi	Bruksanvisning
Informazioni per l’uso	Käyttöinfo
Información de empleo	Brugsanvisning
Instruções de uso	Bruksinformatjon

Made in Germany by 3M ESPE AG Dental Products D-82229 Seefeld - Germany	CE	0123
3M ESPE Dental Products St. Paul, MN 55144-1000	4400725448/01	
3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249.		
3M, ESPE, Alkaliner and Ketac are trademarks of 3M or 3M ESPE AG. © 3M 2003. All rights reserved.		

ENGLISH

Description of Product

Ketac Silver, manufactured by 3M ESPE, is a silver-reinforced glass ionomer cement. On account of its chemical adhesion to the dental enamel and dentin, a gentle preparation and an especially tight seal at the margins is possible. The material can be applied without lining; it releases fluoride ions, and is radiopaque. The fillings are physically to a faint lustre. The restorative material does not corrode and is financially as compatible as glass ionomer restorative material without silver.

For details on all products mentioned below please refer to the corresponding Instructions for Use. The information for use of a product has to be kept for the duration of the application.

Fields of Application

Linings under amalgam fillings
Core build-ups
Uniplanar fillings and dental neck fillings, if aesthetics is not the prime consideration
Primary tooth fillings

Preparation

Remove carious tooth substance only; undercuts are not required. Do not prepare thin bevelling margins. A wall thickness of 0.5 mm has to be secured for the material.

Pulp Protection

In order to prevent pulpal irritation cover areas in close proximity to the pulp with local application of a calcium hydroxide material (e.g. Alkaliner™, manufactured by 3M ESPE).

Conditioning

For optimal chemical bonding to the teeth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner, manufactured by 3M ESPE, to the prepared surfaces and allowing the substance to react for 10 sec.

Then rinse with copious amounts of water.

Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry! **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.

Avoid further contamination.

Dosing

Shake the bottle to loosen the powder. The recommended mixing ratio for powder and liquid is 3.8:1 by weight. This corresponds roughly to one spoon of powder and one drop of liquid. Please keep in mind that the enclosed dosing spoon is a guideline for dosing only! The best way is to bring as much powder as possible in small portions into the liquid up to a viscous-plastic, condensable consistency. Scrape off excess powder on the scoop at the plastic segment.

The liquid bottle package contains a dispensing dropper which replaces the cap. Put the powder and liquid next to each other on a block. While dispensing, the bottle should be held in a vertical position. The drop dispenser has to be free from dried liquid.

Carefully close the bottles with the powder and the liquid after use.

Mixing

Mix Ketac Silver at an ambient temperature of 20° to 25°C/68 to 77°F. Use a cement spatula made of metal or plastic on a mixing pad or a glass slab for mixing. Spread the paste several times until a homogeneous consistency is obtained.

Application

During the application and the setting phase, the working area should be protected from water and saliva. Avoid desiccation of the fresh filling, e.g. by a strong air stream.

When using the material as a build-up under amalgam, the cavity has to be filled in such a way that the dentin is completely covered.

Ketac Silver clings to metal instruments, and for this reason should be washed off with cold water before setting.

Times

The following times apply for an ambient temperature of 23°C/73°F and a relative humidity of 50%:	min : sec
Mixing	0 : 45
Working (including mixing)	4 : 00
Setting time (from the beginning of mix)	7 : 00
Setting time in the mouth	3 : 00

Higher temperatures shorten, lower temperature extend the working time (e.g. mixing on a cooled glass slab). A higher amount of powder also shortens the working time. An over-extended working time will cause the loss of adhesion to the dental enamel and the dentin.

Filling Protection

Glass ionomer goes through two phases of setting, and is susceptible to water during the first setting period. Subsequently, dehydration may be possible, which can be avoided by applying Ketac™ Glaze, manufactured by 3M ESPE, directly after contouring. Apply the glaze to all free surfaces of the glass ionomer cement, and light-cure for 10 sec.

The inhibition layer remaining after the setting of Ketac Glaze may obstruct the setting of the addition-type silicone impression materials. For this reason remove this layer or apply a polyether impression material.

Finishing

Further finishing with rotating instruments should be started 7 min after start of mixing at the earliest. Apply finishing discs or rubber cups under water cooling for fine finishing. If requested, apply Ketac Glaze again for surface sealing and light-cure for 10 sec.

Incompatibility

In susceptible individuals, sensitization to the product cannot be excluded. Use of the product should be discontinued and the product completely withdrawn from contact with the patient, if allergic reactions are observed. There may be rare cases of pulpal sensitivity, especially if the Instructions for Use were not observed.

Storage and Shelf-Life

Protect the powder from moisture. Do not store the product above 25°C/77°F. Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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