## 3M ESPE

## Ketac™ Conditioner

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Made in Germany by
**3M ESPE AG**
Dental Products
D-82229 Seefeld – Germany

**3M ESPE**
Dental Products
St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada
1-800-634-2249.

3M, ESPE, Alkaliner and Ketac are trademarks of 3M or 3M ESPE AG.
© 3M 2004. All rights reserved.

3M ESPE Ketac Conditioner, 30 ml

### ENGLISH

**Product Description**
Ketac Conditioner, manufactured by 3M ESPE, is a polyacrylic acid solution for removal of the smear layer after tooth preparation. Dentin conditioning with Ketac Conditioner significantly increases the bond of glass ionomer cement fillings.

☞ Instructions for Use should not be discarded for the duration of product use.

#### Areas of Application

- Dentin pretreatment prior to filling with glass ionomer cement

#### Precautionary Measures

Avoid exposing mucosa, eyes or skin to Ketac Conditioner. If there is any accidental exposure, rinse exposed surfaces with copious amounts of water.

#### Preparation

- Use a water spray to remove any loose material abraded during preparation, and dry the cavity with air.

#### Pulp Protection

Cover areas in close proximity to the pulp by applying dabs of a hard-setting calcium hydroxide material (e.g. Alkaliner™, manufactured by 3M ESPE) in order to protect the pulp from irritation.

#### Conditioning

- Dispense the conditioner from the bottle onto a pad and apply with a cotton pellet or disposable brush to the smear layer.
- Allow to react for 10 sec.
- Then rinse with copious amounts of water.
- Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry!
  - The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.
- Avoid any contamination by saliva, and commence to fill the cavity without any delay.

#### Incompatibilities

In susceptible individuals, sensitization to the product cannot be excluded. If allergic reactions are observed, use of the product should be discontinued and the product completely removed. In rare cases, pulp complications could occur, especially if the recommendations for processing are disregarded.

#### Storage and Shelf-Life

Reseal bottle immediately after use.

Store the product at 15–25°C/59–77°F.

Do not use after the expiration date.

#### Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

#### Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

#### Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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