<div><b>3M ESPE</b></div> <div><b>Ketac™ Molar Quick Aplicap™</b></div> <div><b>Glass Ionomer Restorative Material</b></div> <div><b>Glasionomer Füllungsmaterial</b></div> <div><b>Ματériau d'obturation en verre ionomère</b></div> <div><b>Materiale per otturazioni vetro-ionomero</b></div> <div><b>Material de obturación de vidrio ionómero</b></div> <div><b>Material de obturção de vidro ionómero</b></div> <div><b>Glasionomeer vulmateriaal</b></div> <div><b>Υαλοίονομερής κοπία για εμφράξεις</b></div> <div><b>Glasjonomer fyllningsmaterial</b></div> <div><b>Lasi-ionomeeritäyteaine</b></div> <div><b>Glasionomer-fyldningsmateriale</b></div> <div><b>Glassionomer fyllningsmateriale</b></div>	
<span></span>	
Instructions for Use	Gebruiksformatie
Gebruiksaanformatie	Οδηγίες χρήσης
Mode d'emploi	Bruksanvisning
Informazioni per l'uso	Käyttöinfo
Información de empleo	Bruksanvisning
Instruções de uso	Bruksinformatjon
<span></span>	
Made in Germany by	
<b>3M ESPE AG</b>	<b>CE</b>
Dental Products	<b>0123</b>
D-82229 Seefeld - Germany	
<b>3M ESPE</b>	
Dental Products	
St. Paul, MN 55144-1000	
3M ESPE Technical Hotline /MSDS Information in U.S.A. and Canada 1-800-634-2249.	
3M, ESPE, Alkaliner, Aplicap, Capmix, Ketac and Rotomix are trademarks of 3M or 3M ESPE AG.	
© 3M 2003. All rights reserved.	

## ENGLISH

**Product Description**

Ketac Molar Quick Aplicap, manufactured by 3M ESPE, is a glass ionomer cement in a capsule. On account of its chemical adhesion to the dental enamel and dentin, a gentle preparation and an especially tight seal at the margins is possible. The product is available in the following shades: A1, A2, A3, A4 and B2. The material can be applied without lining; it releases fluoride ions, and is radiopaque. The dispensable quantity of a capsule is at least 0.1 ml.

For details on all products mentioned below please refer to the corresponding Instructions for Use.

Instructions for Use should not be discarded for the duration of the product use.

**Fields of Application**

Base material for single and multiple surface composite fillings
Core build-ups prior to crown placement
Primary tooth fillings
Single surface fillings in non-occlusal areas
Cervical fillings, if aesthetics is not the prime consideration
Single and multiple surface semipermanent fillings
Extended fissure sealing

**Preparation**

Remove carious tooth substance only; undercuts are not required. Do not prepare thin beveling margins. A wall thickness of 0.5 mm has to be secured for the material.

**Pulp Protection**

To prevent pulpal irritation, the areas around the pulp should be covered by dabbing them with a hardsetting calcium hydroxide preparation (e.g. Alkaliner™, manufactured by 3M ESPE.

**Conditioning**

For optimal chemical bonding to the teeth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner, manufactured by 3M ESPE, to the prepared surfaces and allowing the substance to react for 10 sec.

Then rinse with copious amounts of water.

Blow the cavity dry in only 2–3 short intervals with water and oil-free air or dab dry with cotton pellets. Do not over-dry! **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.

Avoid further contamination.

**Activation Mixing Application**

Use the Aplicap™ activator to activate the capsules for 2 sec. Use a high-frequency mixer with abt. 4,300 oscillations/min, such as Capmix™, or a rotation mixer Rotomix™ for mixing. Use the Aplicap™ applicr, all products manufactured by or for 3M ESPE, for application. Aplicap activator and applicr have been marked in red.

Attention

By all means prevent waiting times between activation, mixing, and application because the beginning of setting of the material will prevent the application. Ketac Molar Quick Aplicap clings to metal instruments, and for this reason should be washed off with cold water before setting.

**Times**

The following times apply for an ambient temperature of 23°C/73°F:

	min <span> </span> :sec
Activation	0 <span> </span> :02
Mixing in Rotomix	0 <span> </span> :08
or	
Mixing in high-frequency mixer e.g. Capmix	0 <span> </span> :10
Working time from start of mixing	1 <span> </span> :40
Setting time from start of mixing	3 <span> </span> :30

Setting is accelerated over 23°C/73°F. An overextended working time will cause the loss of adhesion to the dental enamel and the dentin. Shorter mixing times will extend, longer mixing times will shorten the working and setting times. The mixing times can be chosen between 8 and 15 sec in high-frequency mixers and between 6 and 8 sec in the Rotomix.

**Filling Protection**

Glass ionomers go through two phases of setting, and are susceptible to water during the first setting period. Subsequently, dehydration may be possible, which can be avoided by applying Ketac™ Glaze, manufactured by 3M ESPE, directly after contouring. Apply Ketac Glaze to all free surfaces of the glass ionomer cement, and light-cure for 10 sec. The inhibition layer remaining after the setting of Ketac Glaze may obstruct the setting of the addition-type silicone impression materials. For this reason remove this layer or apply a polyether impression material.

**Finishing**

Further finishing with Arkansas stones, fine-grain diamonds, abrasive discs with successively decreasing grain size or silicone polishers should be started 3.5 minutes after start of mixing at the earliest. If desired, apply Ketac Glaze again for surface sealing and light-cure for 10 sec.

**Incompatibilities**

In susceptible individuals, sensitization to the product cannot be excluded. If allergic reactions occur, use of the product should be discontinued and the product completely removed. In rare cases, particularly when the instructions for processing are not followed, pulp discomfort is possible.

**Storage and Shelf-life**

Store the capsules in the blister packs. Use within 1 month maximum once the blister pack has been opened! Do not store the product above 25°C/77°F.

Do not use after the expiry date.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

**Limitation of Liability**

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or

consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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