3M ESPE
Penta™ Elastomer Elastomer Syringe
Ⓓ Elastomer-Spritze
Ⓕ Seringue à élastomère
Ⓘ Siringa per elastomeri
Ⓔ Jeringa para elásticos
Ⓟ Seringa Elastomera
Ⓝ Elastomeerspuit
Ⓙ Σύριγγα ελαστομερών
Ⓢ Elastomerspruta
Ⓕ Elastomeeri-ruisku
Ⓓ Elastomer sprøjte
Ⓝ Elastomer sprøyte

Instructions for Use	Gebbruksinformatie
Gebrauchsinformation	Οδηγίες χρήσης
Mode d’emploi	Bruksanvisning
Istruzioni per l’uso	Käyttöinfo
Información de uso	Brugsanvisning
Instruções de uso	Bruksinformasjon

Made in Germany by **3M ESPE AG** Dental Products D-82229 Seefeld – Germany

3M ESPE Dental Products St. Paul, MN 55144-1000

3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249

3M, ESPE, Penta and Pentamix are trademarks of 3M or 3M ESPE AG. © 3M 2004. All rights reserved.

ENGLISH

Product Description
The Penta Elastomer syringe is used for intraoral application of light-bodied and medium-bodied consistency impression materials, especially for material dispensed from the Pentamix™ mixer, both products manufactured by 3M ESPE. The translucent syringe barrel allows the operator to determine the amount of material to be dispensed from the mixing device. Syringe barrels can be reused and autoclaved several times. For replacement, spare syringe barrels are available. For details on Pentamix, please refer to the corresponding Instructions for Use.

☞ Instructions for Use should not be discarded for the duration of product use.

Fields of Application

- Application of light-bodied and medium-bodied consistency impression materials

Application

- Remove the syringe plunger from the syringe and attach the Penta Elastomer syringe directly to the mixing tip of the Pentamix unit for filling.
 - The translucent syringe barrel allows the operator to determine the amount of material to be dispensed from the mixing device.
- During or immediately after application dispense all material from the syringe and allow the impression material to set.

Cleaning

- Unscrew the knurled nut and application tip and discard the application tip.
- Push the cured impression material out of the syringe barrel with the syringe plunger, and then clean the syringe with a brush.
 - The translucent syringe barrels can be reused and autoclaved several times.The syringe barrel should be replaced once the surface becomes matte or shows signs of mechanical damage.

- Disassemble the syringe and separately autoclave the individual parts of the syringe. The use of Mucoцит T* (MERZ Dental) is recommended for disinfection.

* Mucoцит T is not available in all countries.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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