3M ESPE
Protemp™ II
Composite for Temporary Crowns and Bridges
Composite für temporäre Kronen und Brücken
Composite pour couronnes et bridges temporaires
Composite per corone e ponti provvisori
Composite para coronas y puentes temporales
Composito para coroas e pontes provisórias
Composiet voor provisorische kronen en bruggen
Σύνθετη ρητίνη για προσωρινές στεφάνες και γέφυρες
Komposit för temporära kronor och broar
Yhdistelmämuovimateriaalia väliaikaisen kruunujen ja siltojen valmistukseen
Komposit til temporære kroner og broer
Komposit til temporære kroner og broer

Instructions for Use	Gebruiksinformatie
Gebruiksaanfichting	Πληροφορίες χρήσεως
Notice d'utilisation	Bruksanvisning
Istruzioni per l'uso	Käyttöohjeet
Informaciones de uso	Bruksanvisning
Instruções de uso	Bruksveiledning

3M ESPE AG	CE	0123
Dental Products		
D-82229 Seefeld - Germany		
3M ESPE		
Dental Products		
St. Paul, MN 55144-1000		
3M ESPE Technical Hotline/MSDS Information in U.S.A. and Canada 1-800-634-2249		
3M, ESPE, Protemp and RelyX are trademarks of 3M or 3M ESPE AG.		
© 3M 2002. All rights reserved.		

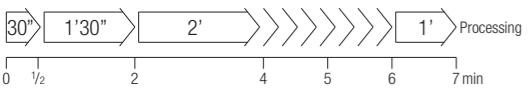
ENGLISH

Product Description
Protemp II, manufactured by 3M ESPE, is a material for the fabrication of temporary restorations inside the patient’s mouth. The 3-component system based on multi-functional methacrylate esters has filling material-like properties. The available colours are A1 extra-bright, A3 bright, and B3 yellow. Instructions for Use should not be discarded for the duration of product use.

Areas of Application
Fabrication of temporary crowns, bridges, inlays, and onlays.

Preparation
Take an alginate or silicone impression of the intact row of teeth. To enhance the strength of the temporary restoration, remove inter-dental grooves. If required due to tight spacing conditions, enlarge the relevant sites by cutting. For easy replacement in the mouth relieve undercut. With missing teeth or gaps in the molar area, cut a groove for a stable bar-shaped connector in the temporary restoration. If required, prior to taking the impression close gaps between front teeth with ready-made teeth acting as spacers; interlock multiple teeth with wax.
Instead of an impression, a laboratory-made deep drawing template or, for single crowns, a shell crown may be used for shaping the temporary restoration.

Times
At room temperature (23°C/73°F) and 50 % humidity, at the recommended mixing ratio of one spindle turn each, the following time periods are available for processing:

Mixing	Fitting the impression and replacement in the mouth	Setting in the mouth	Withdrawal from the mouth	Complete setting		
					Processing	
0	½	2	4	5	6	7 min

Dosing and Mixing
For the fabrication of a temporary crown or bridge unit dispense one full turn each of the dosing syringe spindle of base and catalyst paste onto the mixing block. The locking in position of the spindle after one full turn produces a clearly audible “click” sound. To accelerate the setting reaction, the amount of catalyst may be increased up to two-fold (**max.I**). With increased quantities of catalyst, the processing and withdrawing times are shortened! Further over-dosing or under-dosing changes the mechanical properties of the material (and increases the risk of fracturing).
Take up the catalyst paste with a spatula and mix with base paste by repeatedly streaking out the pastes on the block until the mixture is homogeneous.

Application
Fill the mixture of pastes into the Protemp™ syringe, manufactured for 3M ESPE, and load the dried impression or deep drawing template in the relevant spaces from bottom up. Replace the impression in the mouth. **At the recommended dosages (1:1 turns of the spindle) the material attains a tough-elastic consistency approx. 4 min after the onset of mixing. The material and the impression or deep drawing template must be removed from the mouth within approx. 6 min after the onset of mixing. Control the setting process by means of the excess in the mouth.**

Finishing
Finish the temporary restoration once the material is completely set (not earlier than 7 minutes after the onset of mixing). The superficial inhibition layer formed by the oxygen in air can be removed with organic solvents, e.g. alcohol. Do not inhale polishing dust, use a protective mask or aspiration.

Cementation
Insert temporary restorations with commercial temporary fixation cements, e.g. RelyX™ Temp E or RelyX™ Temp NE, both manufactured by 3M ESPE. Eugenol-containing cements may impair the setting of fixation composites to be used at a later point in time.

Repairs
Fragments of restorations broken-off soon after fabrication may be re-bonded with freshly mixed Protemp II.
With broken temporary restorations that have been in situ for a while, provide for mechanical retention by roughening the surfaces of the breaks and adjacent areas; if required, make undercuts. Light-curing bonding materials can strengthen the bond. Repair the temporary restoration outside the mouth with freshly mixed Protemp II or a methylmethacrylate or, alternatively, use a composite material for repairs inside the mouth.

Cleaning
Paste that has not set can be removed with organic solvents. The application syringe should be cleaned only after the material is completely set.

Notes
Dosing syringes should be carefully re-capped after use.

Incompatibilities
In susceptible individuals, sensitisation to the product cannot be excluded. Use of the product should be discontinued if allergic reactions are observed.

Storage and Stability
Do not store the product above 25°C/77°F. Do not use after the expiry date.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE’s sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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