General Information

RelyX™ Try-In Pastes, manufactured by 3M ESPE, are used to guide the dentist in the selection of the shade required for the final cementation with RelyX™ Veneer Cement, manufactured by 3M ESPE. The shades of RelyX Try-In paste specifically match the final cured shade of RelyX Veneer cement. RelyX Try-In paste contains a polyethylene glycol resin, a zirconia/silica filler and pigments. The RelyX Try-In pastes are water soluble for easy clean up.

Precautions

Use of protective eyewear and gloves are recommended for preventing eye and skin exposure to the product.

Application

1. To prepare the veneer, the internal bonding surface of a veneer surface should be microetched with a hydrofluoric acid etchant. The dental laboratory typically does this. The internal surface of a veneer should be roughened by sandblasting or air abrasion.
2. Remove the temporary restoration, if present, from the tooth. Clean tooth with pumice; rinse thoroughly and lightly air dry.
3. Check the fit and esthetics of the restoration by trying in the veneer with water or the Translucent shade of RelyX Try-In paste. If the fit and esthetics are acceptable, proceed directly to the bonding procedure using the Translucent shade of RelyX Veneer cement.

The veneer is now ready for the application of the ceramic primer and final bonding.

Storage and Use

1. RelyX Try-In pastes are designed to be used at room temperature of 21-24°C or 70-75°F.
2. Shelf life in a refrigerator or at room temperature is 24 months. See outer package for expiration date. Storage in refrigerator ensures longest possible shelf life. Allow paste to come to room temperature before use.
3. Do not expose the paste to elevated temperatures.


No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

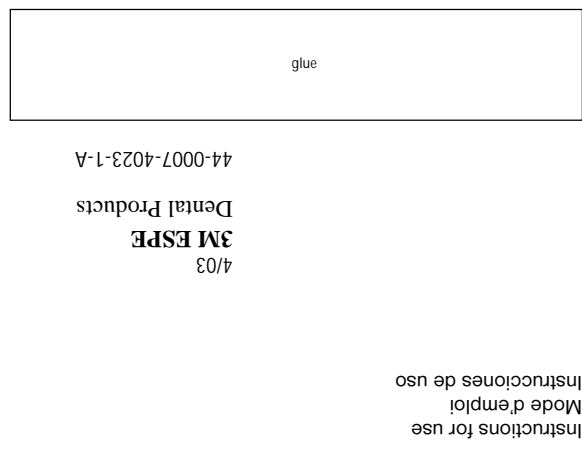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RelyX™ Try-In Paste

Try-In Paste

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4. If color adjustment is needed, select the appropriate shade of RelyX Try-In paste. Apply RelyX Try-In paste directly onto the internal surface of the veneer. Gently seat the veneer onto the tooth preparation. With multiple veneers, start at the midline and seat veneers sequentially working back. The shade of RelyX Try-In paste is formulated to precisely match the corresponding cured shade of the cement.

Once the fit and esthetics are verified, remove the veneers and thoroughly rinse the RelyX Try-In paste from the tooth and restoration with a water spray and then dry.

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3M, ESPE, and RelyX are trademarks of 3M or 3M ESPE AG.
© 3M 2003. All rights reserved. 44-0007-4023-1-A