General Information

P-10™ Resin Bonded Ceramic, manufactured by 3M ESPE, is a radiolucent, two paste posterior filling material recommended for Class I and Class II restorations. The filler in P-10 is quartz. The filler loading is 74% by volume with a particle size range of 20 to 0.3 microns.

Precautions For Dental Personnel and Patients**P-10 Paste Precautions**

P-10 paste contains BIS-GMA and TEGDMA Resins. A small percentage of the population is known to have an allergic response to acrylate resins. To reduce the risk of allergic response, minimize exposure to these materials. In particular, exposure to uncured resins should be avoided. **Use of protective gloves and a no-touch technique is recommended.** If skin contact occurs, wash skin with soap and water. Acrylates may penetrate commonly used gloves. If restorative material contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If accidental contact with eyes or prolonged contact with oral soft tissue occurs, flush with large amounts of water. If irritation persists, consult a physician.

Instructions for Class II Placement

- Prophy.** Use a slurry of plain pumice and water to clean tooth.
- Isolation.** Rubber dam is the preferred method of isolation.
- Wedging.** Firmly insert wedges interproximally to provide early separation of teeth.
- Cavity Preparation.** Remove caries and/or old restoration with minimal tooth reduction and finish with rounded internal line angles. **Bevel enamel cavosurface margins, except where minimal enamel is present at gingival seat.**
- Pulp Protection.** In deep preparations, the pulpal floor should be coated with a hard-setting calcium hydroxide base restricting it to the deeper portion of the cavity, leaving dentin and enamel walls free for bonding.

6. **Matrix Placement.** Remove wedges. Place a thin dead-soft matrix band and insert new wedges firmly. Burnish the matrix band to establish proximal contour and contact area. Adapt the band to seal the gingival area and avoid overhang.

Note: For some cases, it may be preferred to place the matrix after the enamel etching and adhesive application steps.

- Etch** according to the instructions of the Scotchbond™ adhesive product, manufactured by 3M ESPE, that is used.
- Apply and cure** adhesive according to the instructions of the Scotchbond adhesive product that is used.

9. **Mixing and Placement of P-10 Material.** Place equal amounts of paste A and paste B on a mixing pad using opposite ends of mixing spatula. Spatulate thoroughly for a full 30 seconds, continuously folding the material onto itself, to ensure a homogeneous mix.

For equal quantities of paste A and paste B used at 22°C (72° F) room temperature conditions:

- Mixing Time 30 seconds
- Working Time 1 - 1 1/2 minutes
- Hardening Period 4 minutes
- Total Time 6 minutes

Working Time Adjustment

The working time of P-10 can be adjusted to the dentist's needs by altering the ratio of the two pastes. Ratios up to 2:1 can be used. To extend placement time, use more of paste B than paste A; to shorten, use more of paste A than paste B.

- Place mixed P-10 material into cavity with a placement instrument or with a syringe. To prevent air voids, add in small increments with the placement instrument or begin on cavity floor with syringe.
- Fill to excess and condense with appropriate instrument.

- When material on pad begins to gel, do not add more to cavity or attempt to manipulate what is already in cavity. Use a second mix if necessary.
- Allow to polymerize undisturbed. **Six minutes from the start of mix and four minutes after material on pad is set hard, wedges and matrix can be removed and contouring and finishing begun.**

10. **Finishing.** Contour restoration surfaces with fine finishing diamonds, burs or stones. Contour interproximal surfaces with 3M ESPE Finishing Strips, manufactured for 3M ESPE.

11. **Adjusting Occlusion.** Check occlusion with thin articulating paper. Centric and lateral excursion contacts should be examined. Carefully adjust occlusion by removing material with a fine finishing diamond or stone.

12. **Polishing.** Polish with Sof-Lex™ discs, manufactured by 3M ESPE, and strips and with white stone or rubber points where discs are not suitable.

Additional Notes

- Sensitivity - some patients may experience transitory postoperative sensitivity. The risk of sensitivity can be minimized by the following measures:
 - Remove minimal tooth structure.
 - Use proper isolation. Use of a rubber dam is highly recommended.
 - Adequate pulp protection. Use a glass ionomer cavity liner/base on appropriate dentinal surfaces.
 - Adjust occlusion carefully. Check for hyper-occlusion, particularly in lateral excursion contacts.

Storage and Use

- Do not expose materials to elevated temperature.
- Do not store materials in proximity to eugenol-containing products.
- This system is designed to be used at room temperature of approximately 21-24° C or 70-75° F.
- Shelf life at room temperature is 36 months. See outer package for expiry date.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability

Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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