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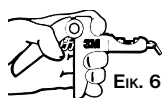
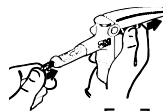
3M ESPE AG
Dental Products

D-82229 Seefeld – Germany

3M and ESPE are trademarks of 3M or 3M ESPE AG.
© 3M 2002. All Rights reserved. 44-0023-3936-2-A

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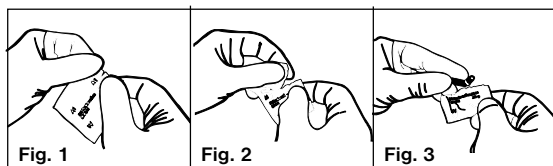
8/02

1. 2. Open foil. 3. Remove capsule.

4. Insert capsule into dispenser manufactured for 3M ESPE. Open dispenser handle as wide as possible. Pull plunger toward opened handle. Place capsule tip into the barrel slot.

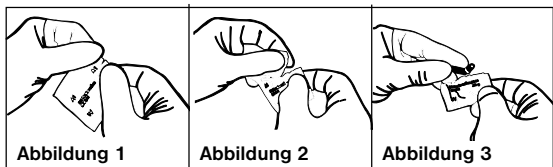
5. Seat the capsule in the barrel. Push capsule forward (away from handle) as far as possible. Push plunger towards capsule. Close handle grip until plunger tip contacts capsule. Remove the cap from the capsule.

6. Dispense the material into the cavity preparation using a slow, steady pressure. Excessive force is contraindicated. The capsule may be rotated 360° to obtain an appropriate entrance into the cavity preparation.

7. Remove the used capsule. Open the dispenser handle as wide as possible. Pull plunger away from capsule. Push the capsule toward the plunger. Turn dispenser upside down to drop capsule into hand.

Disinfection Recommendations. The 3M ESPE dispenser and plunger have not shown any visible signs of degradation following repeated cycling in an autoclave (at temperatures not exceeding 285°F/140°C) or repeated room temperature soaking cycles in concentrated glutaraldehyde or concentrated phenolic glutaraldehyde solutions. Consult individual autoclave or disinfectant used instructions for sterilizing and disinfection.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

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