

Filtek™ Supreme Plus Universal Restorative Adper™ Adhesive Systems Sof-Lex™ Discs and Strips

Preparation:

- Select tooth shade using the VITAPAN® classical shade guide.
- Select composite shades using the 3M™ ESPE™ Filtek™ Supreme Plus Shade Selector Wheel as a guide.
- Prepare the tooth.
- Bevel enamel margins.



Adhesive placement:

- Apply adhesive following directions for the 3M™ ESPE™ Adper™ Adhesive System of choice.



Place composite – dentin replacement:

- Place and cure Dentin shade of 3M™ ESPE™ Filtek™ Supreme Plus Restorative to provide opacity and color intensity.
- Follow increment depth and cure time recommendations for chosen shade.



Place composite – dentin replacement:

- Place and cure Body shade of Filtek Supreme Plus to the dentin enamel junction.
- Follow increment depth and cure time recommendations for chosen shade.




**Filtek™ Supreme Plus Universal Restorative
Adper™ Adhesive Systems
Sof-Lex™ Discs and Strips**



Place composite – enamel replacement:

- Fill remaining preparation using Enamel and/or Translucent shades of Filtek Supreme Plus restorative to provide translucency for depth of color and incisal edge match.
- Follow increment depth and cure time recommendations for chosen shade.

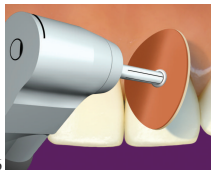

Finish and Polish:

- Use 3M™ ESPE™ Sof-Lex™ Finishing and Polishing discs to finish the restoration.
- The aesthetic result will be enhanced by creating surface texture similar to the adjacent teeth.



Finish and Polish:

- Use 3M™ ESPE™ Sof-Lex™ Strips for finishing proximal areas by gently inserting the center gapped area between teeth. Operating sequence of strips (C/M, F/SF) is the same as the discs.

2 of 2

3M ESPE Customer Care 1-800-634-2249
Please refer to instructions for more detailed information
as well as precautionary and warranty information.

VITAPAN® is a trademark of VITA

Zahnfabrik H. Rauter GmbH & Co.KG

3M, ESPE, Adper, Filtek and Sof-Lex are trademarks of 3M or 3M ESPE AG.

Used under license in Canada.

44-0007-4305-2

70-2009-3729-3

© 2005 3M