

## 3M™ ESPE™ F2000 Compomer Primer/Adhesive in the 3M™ ESPE™ Clicker Dispenser

### Precautions:

- Do not depress the dispenser lever during cap removal and/or replacement.



- Do not advance the dispenser plunger during cap removal and/or replacement.



### Operation:

- Remove cap.
- Press and hold cap lever to unlock cap.



- Slide cap off dispenser.



### Dispense:

- Touch cartridge tips to bottom of well.  
**Contact of the tips with the well bottom is critical to assure proper dispensing.**



## F2000 Compomer Primer/Adhesive in the 3M™ ESPE™ Clicker™ Dispenser



6

- Fully depress lever to dispense material from each side of cartridge into well.
- Release lever.
- Remove dispenser from well.



7

### Clean:

- Wipe the cartridge tips with an alcohol-dampened gauze.



8

### Replace Cap:

- Hold the sides of the cartridge and slide cap into place until securely latched as indicated by an audible click.