

3M™ ESPE™ F2000 Compomer Restorative with 3M™ ESPE™ F2000 Compomer Primer/Adhesive in the 3M™ ESPE™ Clicker Dispenser

Indication:

- Crown buildup

Preparation:

- Isolate the tooth.
- Remove caries.
- Place retention if multiple cusps are missing.
- Place appropriate matrix if desired.
- The blue shade will provide contrasting color to tooth structure.

Dispense:

- Dispense 3M™ ESPE™ F2000 Compomer primer/adhesive into the mixing well.
- Using an applicator brush, mix solution in the well.



Prime/Bond:

- Generously apply F2000 compomer primer/adhesive to enamel and dentin.
- Wait 30 seconds. **Keep surfaces wet for the full time.**
- Gently dry for 5-10 seconds.



- Light-cure the compomer primer/adhesive for 10 seconds.



F2000 Compomer Restorative with F2000 Compomer Primer/Adhesive in the Clicker Dispenser



4

Place Compomer:

- F2000 compomer will not self-cure.
- Place F2000 compomer incrementally.
- Light-cure each increment for 40 seconds.



5

Prepare Crown:

- The F2000 compomer core buildup can be prepared immediately after light-curing.



6

Notes:

- The compomer core is compatible with commonly used impression materials.
- The compomer core should be kept wet with saliva or lubricated to prevent bonding to chemical-cure provisionals.
- Commonly used temporary luting cements will not bond to the compomer core.