

## 3M™ ESPE™ F2000 Compomer Restorative F2000 Primer/Adhesive in the Clicker™ Dispenser Sof-Lex™ Discs and Strips

### Indications:

- Class V cavities & cervical erosion/abrasion lesions
- Root caries lesions
- Class III cavities
- Class I & II cavities in primary teeth
- Class I & II cavities in permanent teeth provided the width of the cavity is less than two thirds of the intercuspal distance
- Temporary repair of fractured teeth

### Preparation:

- Select shade using shade guide.
- Isolate the tooth.
- Remove caries.
- If no preparation is made, clean surfaces with pumice slurry.

### Dispense:

- Dispense 3M™ ESPE™ F2000 compomer primer/adhesive into the mixing well.
- Using an applicator brush, mix solution in the well.



### Prime/Bond:

- Generously apply F2000 compomer primer/adhesive to enamel and dentin.
- Wait 30 seconds. **Keep surfaces wet for full time.**
- Dry **gently** for 5-10 seconds.



## F2000 Compomer Restorative F2000 Primer/Adhesive in the Clicker Dispenser Sof-Lex™ Discs and Strips



- Light-cure the compomer primer/adhesive for 10 seconds.



### Place Compomer:

- Place F2000 compomer restorative in increments.
- Light-cure each increment for 40 seconds.



### Finish/Polish:

- Use the Sof-Lex Finishing and Polishing System for finishing and polishing the restoration.