

3M™ ESPE™ F2000 Compomer Restorative Adper™ Single Bond Dental Adhesive System Sof-Lex™ Discs and Strips

Indications:

- Class V cavities & cervical erosion/abrasion lesions
- Root caries lesions
- Class III cavities
- Class I & II cavities in primary teeth
- Class I & II cavities in permanent teeth, provided the width of the cavity is less than two-thirds of the intercuspal distance.
- Temporary repair of fractured teeth

Preparation:

- Select shade using shade guide.
- Isolate the tooth.
- Remove caries.
- If no preparation is made, clean surfaces with pumice slurry.

Etch:

- Apply Scotchbond™ Etchant to enamel and dentin. Wait 15 seconds.
- Rinse.
- **Blot excess water leaving tooth moist.***



Bond:

- Using a fully saturated brush tip for each coat, apply 2 **consecutive coats** of Adper™ Single Bond Adhesive to enamel and dentin.
- Dry **gently** for 2-5 seconds.
- Light-cure the adhesive for 10 seconds.



*Note: A 3M™ ESPE™ Mini-Sponge (No. 7522S) or a moist cotton pellet may be used for blotting.

F2000 Compomer Restorative Adper™ Single Bond Adhesive System Sof-Lex™ Discs and Strips



4

Place Compomer:

- Place F2000 Compomer restorative in increments.
- Light-cure each increment for 40 seconds.



5

Finish/Polish:

- Use the Sof-Lex™ Finishing and Polishing System for finishing and polishing the restoration.