

Elipar™ FreeLight 2 LED Curing Light

Removing the Battery Cap:

- Rotate the cap at the bottom of the handpiece counter-clockwise until it stops, and then remove the cap.



Inserting the Battery:

- Insert the rechargeable battery in a fluent motion as directed by the arrow until you hear or feel the battery **SNAP** into place.
- Replace the cap and rotate clockwise until it locks into place.
- Malfunction of the light will result if the battery has not been seated fully and secured into the handpiece.
- In the event of malfunction, remove the battery from the handpiece as instructed below.



Removing the Battery:

- Push a small spatula or similar aid between the cylinders of the battery until the snap-lock releases the battery.



- Remove the battery as illustrated in figure 4. Reinsert the battery or insert a new battery and replace the cap as described and illustrated in figure 2 above.



Elipar™ FreeLight 2 LED Curing Light



Charging Steps:

- Place the charger base on a level surface. Do not obstruct the vent slots on the bottom of the unit.
- Plug in the charger base. The green LED on the left side of the device should illuminate indicating the unit is ready for operation.
- Prior to the first use, place the handpiece in the charger base to charge the new battery.
- The yellow LED on the left side of the charger base will illuminate after approximately 2 seconds. The battery is charged once the yellow LED turns off.
- **Note: it will take 3 TO 4 charging cycles (completely discharged and recharged) before the battery reaches maximum charging capacity.**



Cleaning/Disinfecting the Charger Base and Handpiece:

- **Wipe handpiece and charger base for disinfection. Do not spray. Disinfecting agents must not enter openings on unit.**
- Apply disinfecting agent with a soft cloth and allow disinfectant to remain on surface for the amount of time recommended by the manufacturer. Follow precautions for use.
- Perform a final wipe with a clean cloth to remove any residual deposits of the disinfectant solution.
- **Wet charge contacts will shut down the charging process and will be signaled by ongoing beeps and a flashing yellow light on the charging base.** If this occurs, simply remove handpiece from charging base and thoroughly dry contacts on charging base and handpiece as described above.
- Do not use solvents or abrasive cleaning agents as these may damage plastic parts.



Elipar™ FreeLight 2 LED Curing Light

Cleaning/Sterilizing the Light Guide:

- Cured composite on the tip of the light guide should be removed with alcohol. A plastic spatula may help in removing the material. Do not use any sharp or pointed instruments.
- The light guide can be steam autoclaved. Do not sterilize with dry heat or chemicals.
- Water spots should be wiped off both ends of the light guide before and after steam sterilization.

