

## 3M™ ESPE™ F2000 Compomer Restorative with 3M™ ESPE™ F2000 Compomer Primer/Adhesive in the 3M™ ESPE™ Clicker™ Dispenser Sof-Lex™ Discs, Strips and Finishing Brush

### Indications:

- Class I & II cavities in primary teeth
- Class III & V cavities
- Temporary repair of fractured teeth

### Preparation:

- Select shade using shade guide.
- Isolate the tooth.
- Remove caries.
- If no preparation is made, clean surfaces with pumice slurry.

### Dispense:

- Dispense the 3M™ ESPE™ F2000 Compomer Primer/Adhesive into the mixing well.
- Using an applicator brush, mix solution in the well.



### Prime/Bond:

- Generously apply F2000 compomer primer/adhesive to enamel and dentin.
- Wait 30 seconds. **Keep surfaces wet for full time.**
- Dry **gently** for 5-10 seconds.
- Light-cure the compomer primer/adhesive for 10 seconds.



## F2000 Compomer Restorative with F2000 Compomer Primer/Adhesive in the Clicker™ Dispenser Sof-Lex™ Discs, Strips and Finishing Brush



4

### Place Compomer:

- Place F2000 compomer restorative in increments.
- Light-cure each increment for 40 seconds.



5

### Finish/Polish:

- Finish occlusal surface using a Sof-Lex Finishing Brush.
- Finish proximal surfaces with Sof-Lex Extra Thin Discs and Sof-Lex Strips.