


Filtek™ Supreme Plus Universal Restorative Vitrebond™ Light-Cure Glass Ionomer Liner/Base Adper™ Adhesive Systems Sof-Lex™ Discs, Strips and Finishing Brush

- Select shade using the VITAPAN® classical shade guide and the 3M™ ESPE™ Filtek™ Supreme Plus Shade Selector Wheel.
- Prepare the tooth and isolate.

Apply liner/base if desired:

- Mix a level scoop of 3M™ ESPE™ Vitrebond™ Powder with 1 drop of Vitrebond Liquid on a mix pad.
- Apply a thin layer of the liner/base to dentin surfaces.
- Light-cure 30 seconds.

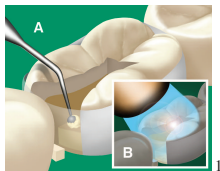

Adhesive placement:

- Apply adhesive following directions for 3M™ ESPE™ Adper™ adhesive system of choice.



Place composite – proximal box:

- To ensure good adaptation, place chosen Dentin (or Body) shade of Filtek Supreme Plus restorative in 1mm increments in the proximal box and on the floor of the cavity.
- Follow increment depth and cure time recommendations for chosen shade.




Place composite – dentin replacement:

- Continue placing chosen Body (or Dentin) shade of Filtek Supreme Plus restorative in increments to the dentin enamel junction.
- Use remaining coronal tooth structure as a guide for placement and adaptation.
- Follow increment depth and cure time recommendations for chosen shade.



**Filtek™ Supreme Plus Universal Restorative
Vitrebond™ Light-Cure Glass Ionomer Liner/Base
Adper™ Adhesive Systems
Sof-Lex™ Discs, Strips and Finishing Brush**

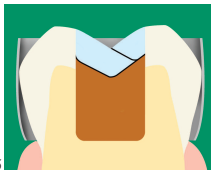

Place composite – enamel replacement:

- Place chosen Enamel and/or Translucent shade of Filtek Supreme Plus restorative in increments.
- Use remaining coronal tooth structure as a guide for placement and adaptation. *Overfill slightly.*
- Follow increment depth and cure time recommendations for chosen shade.



Finish and Polish:

- After contouring posterior composite, polish surface with 3M™ ESPE™ Sof-Lex™ finishing brush at low speeds.
- Use 3M™ ESPE™ Sof-Lex™ Sof-Lex Strips and Discs for finishing and polishing proximal areas.



Occlusion:

- Check lateral and centric occlusion.
- Adjust as necessary.

2 of 2

3M ESPE Customer Care 1-800-634-2249
Please refer to instructions for more detailed information
as well as precautionary and warranty information.

VITAPAN® is a trademark of VITA

Zahnfabrik H. Rauter GmbH & Co.KG

3M, ESPE, Adper, Filtek, Vitrebond and Sof-Lex are trademarks of 3M or 3M ESPE AG.

Used under license in Canada.

44-0007-4307-8

70-2009-3731-9

© 2005 3M