


Filtek™ Supreme Plus Universal Restorative Vitrebond™ Light-Cure Glass Ionomer Liner/Base Adper™ Adhesive Systems Sof-Lex™ Discs, Strips and Finishing Brush

- Select shade using the VITAPAN® classical shade guide.
- Prepare the tooth and isolate.

Apply liner/base if desired:

- Mix a level scoop of 3M™ ESPE™ Vitrebond™ Powder with 1 drop of Vitrebond Liquid on a mix pad.
- Apply a thin layer of the liner/base to dentin surfaces.
- Light-cure 30 seconds.

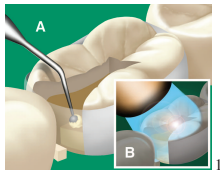


Adhesive placement:

- Apply adhesive following directions for 3M™ ESPE™ Adper™ adhesive system of choice.



Place composite:

- Place chosen Body shade of 3M™ ESPE™ Filtek™ Supreme Plus restorative in increments.
- Use remaining coronal tooth structure as a guide for placement and adaptation.
- Follow increment depth and cure time recommendations for chosen shade.

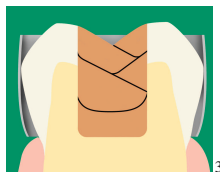

Finish and Polish:

- After contouring posterior composite, polish surface with 3M™ ESPE™ Sof-Lex™ Finishing Brush at low speeds.
- Use 3M™ ESPE™ Sof-Lex™ Strips and Discs for finishing and polishing proximal areas.



**Filtek™ Supreme Plus Universal Restorative
Vitrebond™ Light-Cure Glass Ionomer Liner/Base
Adper™ Adhesive Systems
Sof-Lex™ Discs, Strips and Finishing Brush**



Occlusion:

- Check lateral and centric occlusion.
- Adjust as necessary.

2 of 2

3M ESPE Customer Care 1-800-634-2249
Please refer to instructions for more detailed information
as well as precautionary and warranty information.

VITAPAN® is a trademark of VITA

Zahnfabrik H. Rauter GmbH & Co.KG

3M, ESPE, Adper, Filtek, Vitrebond and Sof-Lex are trademarks
of 3M or 3M ESPE AG. Used under license in Canada.

44-0007-4306-0

70-2009-3730-1

© 2005 3M