

RelyX™ ARC Adhesive Resin Cement

Indications:

This system is indicated for adhesive (bonded) amalgam restorations. Prepare a standard amalgam preparation. Roughen any existing amalgam or composite. Treat existing amalgam or composite using the same procedure outlined below for enamel and dentin. When bonding amalgam rub the inner surface of the matrix band with wax to prevent inadvertent bonding.

Etchant:

- Apply 3M™ ESPE™ Scotchbond™ Etchant to enamel and dentin - wait 15 seconds.
- Rinse for 10 seconds.
- **Blot excess water leaving tooth moist.***



Adhesive:

- Apply 2 consecutive coats of 3M™ ESPE™ Adper™ Single Bond Adhesive to enamel and dentin.
- Dry gently for 2-5 seconds.



Light-Cure:

- Light-cure for 10 seconds.



Apply Cement:

- Triturate amalgam while applying cement to the preparation.
- Dispense cement onto a mixing pad and mix for 10 seconds.
- Using a brush, apply cement to the preparation.



RelyX™ ARC Adhesive Resin Cement



Completing the Restoration:

- Condense and burnish the amalgam.

5