

RelyX™ ARC Adhesive Resin Cement

Indications:

This system is indicated for cementing endodontic posts. Paper points are recommended both for applying solutions and for removal of excess material after application. Trial fit and adjust post as needed.

Etching:

- Apply Scotchbond™ Etchant to the canal — wait 15 seconds.
- Rinse for 10 seconds.
- Dry for 2 seconds. **Remove excess moisture with a paper point.**



Adhesive:

- Apply Adper™ Single Bond Plus Adhesive to the canal.
- Dry for 5 seconds.
- **Remove excess (if any) with a dry paper point.**



Light-Cure:

- Light-cure for 10 seconds.
- **Try-in post to insure fit.**



Apply Cement:

- Dispense cement onto a mixing pad and mix for 10 seconds.
- Apply cement in and around canal using an appropriate instrument.
- Place a thin layer of mixed cement on post.



RelyX™ ARC Adhesive Resin Cement

**Seat The Post and Light-Cure:**

- Seat the post.
- Begin clean-up of excess cement 3-5 minutes after seating.
- Light-cure for 40 seconds from an occlusal direction. *The remaining cement will set via the self-cure reaction.*
- Core material can now be bonded to the post and tooth structure.