

RelyX™ Luting Cement

Indications:

- For permanent cementing of:
 - Porcelain-fused-to-metal crowns and bridges
 - Metal inlays, onlays, crowns and bridges
 - Crowns made with all-alumina or all-zirconia cores such as Procera® AllCeram
 - Pre-fabricated and cast endodontic posts
- Cementing orthodontic bands

Tooth Preparation:

- Remove provisional restoration and all temporary cement.
- Clean tooth with oil-free pumice paste, rinse, and dry.
- DO NOT DESSICATE THE TOOTH.

Mixing:

- Use an equal number of powder scoops and liquid drops.
- Fluff the powder. Dispense 3 level scoops for crown, 6 for bridge.
- Hold liquid vial vertically and dispense 3 drops for crown, 6 for bridge. Keep the vial tip clean.
- Mix all the powder into the liquid, about 30 seconds.



Load the crown/bridge:

- Spread the cement over all the internal surfaces of the crown/bridge.
- Working time is 2.5 minutes.



Placement:

- Seat the crown/bridge.
- Wait at least 3 minutes after placement.
- Remove excess cement with an appropriate instrument.



1 of 1