

RelyX™ Luting Plus Cement

Indications:

For permanent cementation of:

- Porcelain-fused-to-metal crowns and bridges
- Metal crowns, bridges, inlays and onlays (including prefabricated provisional metal crowns).
- All-zirconia or all-alumina strengthened core restorations
- Prefabricated or cast endodontic posts

Cementing orthodontic bands

Tooth Preparation:

- Remove provisional restoration and all temporary cement.
- Clean tooth with oil-free pumice paste, rinse and lightly dry leaving tooth surface moist.
- **DO NOT OVERDRY THE TOOTH**


Dispensing and Mixing:

- Remove cap and dispense desired amount of cement onto mixing pad. 1-2 “Clicks” of material are sufficient for a single tooth restoration.
- **Note: To avoid water evaporation and drying out of the cement pastes, dispense and mix cement immediately prior to use.**
- Replace cap securely onto Clicker™ dispenser until an audible click is heard.
- Mix cement with spatula for 20 seconds.



Load Cement Into Restoration:

- Working time of mixed cement is 2.5 minutes.
- Apply thin layer of cement to the inside surface of the restoration.

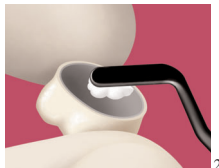

RelyX™ Luting Plus Cement



Placement of Restoration:

- Seat restoration with light pressure.
- Excess cement can be removed after 2 minutes in the mouth with an appropriate instrument (e.g. explorer, scaler).
- Set time is 5 minutes after placement in the mouth.
- Finish restoration and adjust occlusion.