

Protemp™ 3 Garant™ Temporization Material

Prior to tooth preparation, take the impression using Position™ Penta™ Quick Impression Material or fabricate the matrix.

Insert the 10:1 plunger into the Garant 4:1/10:1 dispenser. Slide the cartridge downward into the front of the dispenser and lock into place by pressing the locking lever downward. **Note: Bleed cartridge prior to first use.**



Attach a mixing tip and turn clockwise to lock into position.



Bleed through mix tip. **Note: Prior to each application, bleed a small amount (pea-sized) through the mix tip before dispensing into matrix.**



(A) Dispense material into the impression or matrix. **Working time 50 seconds.**
(B) Seat in the mouth and hold firmly in place.



Remove from mouth. Earliest removal from mouth after 1:35 Minutes (from start of mix). If desired, it may remain in mouth for as long as 2:30 minutes.



Protemp™ 3 Garant™ Temporization Material



Trim during flexible stage after 1:35 minutes (from start of mix). Trim excess material and proximal undercuts with crown scissors.




Finish and polish after 5:00 minutes (from start of mix). When restoration is completely set, remove oxygen-inhibited layer with ethanol, finish using fine carbide burs and polish with 3M™ ESPE™ Sof-Lex™ Discs and Brush.



Cement provisional using 3M™ ESPE™ RelyX™ Temp E or RelyX™ Temp NE Temporary Cement. Note: Before cementation of permanent restoration RelyX Temp E must be thoroughly removed, as eugenol residues can inhibit the polymerization of luting composites.

Time Table

(Temperature: 23° C)

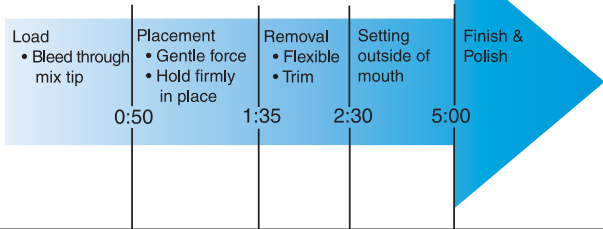

2 of 2