

Imprint™ II Impression Material Systems

Apply 3M™ ESPE™ Tray Adhesive; dry a minimum of 5 minutes.



Locate the V-Notch on the cartridge flange. The V-Notch must be facing down as the impression material cartridge is loaded into the dispenser.



Insert the cartridge flange directly behind the retainers on the dispenser and close the cartridge lock.



To remove the cartridge cap, rotate it 90° in a counter-clockwise direction (A). With firm pressure, tilt the cap downward and peel it away from the cartridge (B).



Bleed the cartridge.



Imprint™ II Impression Material Systems




Locate and align the V-Notch on the mix tip with the V-Notch between the cartridge barrels.

Tips



To attach the mix tip, hold the mix tip **collar** and push firmly to ensure that it is flush with the cartridge (A). Next, rotate the collar 90° in a clockwise direction to its locked position (B).

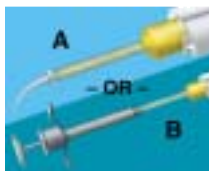

Attach an intraoral tip to the wash material mix tip (A) or load an intraoral syringe (B).

Important Information:

Avoid contact with chemicals known to inhibit the set of vinyl polysiloxane materials such as latex rubber, acrylic and methacrylate residues.

To avoid inhibition caused by temporary materials, **the final impression should be made before fabricating the provisional crown or bridge.**

2 of 2