

Imprint™ II Garant™ Monophase Impression Material System

Fill the tray with Imprint II Garant Monophase Material.




Syringe monophase impression material around the clean, dry tooth preparation(s). **Intraoral Working Time: 1 minute.**



Slowly seat the tray in the mouth. **The tray must be seated within 1 minute of the start of oral syringing and within 2 minutes of the start of tray filling.**



Immobilize the tray using passive pressure. **Oral Setting Time: 4:00 minutes.**

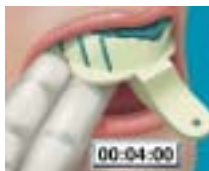

Remove the impression, rinse, dry and disinfect. The stone model may be poured after 30 minutes.

