

## Imprint™ II Garant™ Heavy Body/Light Body Impression Material System

Syringe light body material around the clean, dry tooth preparation(s) using either Imprint II Garant Light Body (A) or Imprint II Garant Regular Body (B).

**Intraoral Work Time: 1 minute.**



Fill the tray with Imprint II Garant heavy body while the light body impression material is being syringed on the preparation.



Slowly seat the tray in the mouth.



Immobilize the tray using passive pressure. **Oral Set Time: 4 minutes.**



Remove the impression, rinse, dry and disinfect. The stone model may now be poured.

