

## Imprint™ II Impression Material

- Apply VPS Tray Adhesive: dry a minimum of 5 minutes.



- Insert the loaded Penta cartridge into the 3M™ ESPE™ Pentamix™ Mixing Unit.



- For initial use extrude a small amount of material and discard it. Wait for a uniform mix.



- Locate the V-Notch on the cartridge flange. The V-Notch must be facing down as the cartridge is loaded into the 3M™ ESPE™ Garant™ Dispenser.



- To remove the cartridge cap, rotate it 90° in a counter-clockwise direction (A).
- With firm pressure, tilt the cap downward and peel it away from the cartridge (B).



## Imprint™ II Impression Material




- Bleed the cartridge.



- Locate and align the V-Notch on the mix tip with the V-Notch between the cartridge barrels.



- To attach the mix tip, hold the mix tip collar and push firmly to ensure that it is flush with the cartridge (A).
- Rotate the collar 90° in a clockwise direction to its locked position (B).

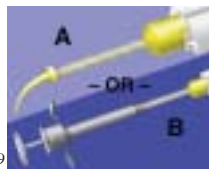

- Attach an intraoral tip to the light body material mix tip (A) or load an intraoral syringe (B).



- Syringe light body material around the clean, dry tooth preparation(s) using either 3M™ ESPE™ Imprint™ II Garant™ Light Body (A) or 3M™ ESPE™ Imprint™ II Garant™ Regular Body (B).

## Imprint™ II Impression Material

- Begin filling the tray.




- Slowly seat the tray in the mouth.



- Immobilize the tray using passive pressure. Oral Set Time: 4 minutes.



- Remove the impression, rinse, dry and disinfect. Wait 30 minutes to pour the stone model.

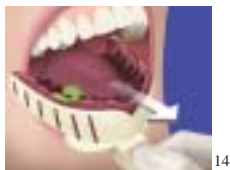

- 3M ESPE recommends horizontal storage of the ready-for-use cartridges.



## Imprint™ II Impression Material

---

### Important Information:

Imprint II cartridges are specifically designed for use with the Garant™ Dispenser.

- Avoid contact with chemicals known to inhibit the set of vinyl polysiloxane materials such as latex rubber, acrylic and methacrylate residues.
- To avoid inhibition caused by temporary materials, the final impression should be made before fabricating the provisional crown or bridge.