

Imprint™ II Garant™ Quick Step Heavy Body/Light Body Impression Material System

Syringe light body impression material around the clean, dry tooth preparation(s) using either Imprint II Garant Quick Step Light Body (A) or Imprint II Garant Quick Step Regular Body (B) Viscosity Wash.

Intraoral Working Time: 40 seconds




Fill the tray with Imprint II Garant Quick Step heavy body material while the light body material is being syringed on the preparation.



Slowly seat the tray in the mouth. **The tray must be seated within 40 seconds of the start of oral syringing and within 1:15 minutes of the start of tray filling.**



Have patient bite down into centric occlusion. **Oral Setting Time: 2:30 minutes**

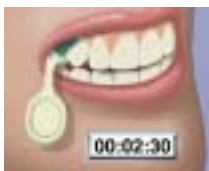

Remove the impression, rinse, dry and disinfect. The stone model may be poured after 2 hours.

