

Paradigm™ C Glass Ceramic Block for CEREC®

Selection of the block size for the restoration is automatically determined upon completion of all of the design steps required. You will be prompted by the CEREC machine to insert a block corresponding to the size of the restoration. Proceed with the following size guidelines.

3M™ ESPE™ Paradigm™ C Block is made in five sizes I8, I10, I12, I14, & V5-12; these correspond to the CEREC sizes.



Paradigm™ C
Glass Ceramic Block for CEREC

Milling Lubricant

Either Sirona™ Dentatec™ or Sirona™ Dentatec ProCAD CEREC Milling Concentrate can be used with Paradigm C blocks. The recommended level of concentrate 50 to 75 ml can be used. It is important to change the water as indicated by the CEREC software.

Polishing and Characterization

Polish the cemented restoration, using the polishing brushes and pastes commonly used for ceramics.

Use Paradigm C Stain/Glaze and Paradigm C Glaze liquid for staining restorations.